

## New Improved ColdSPOT<sup>®</sup> with Floral Design Cold Comfort for the Post-Biopsy Patient

Beekley introduces the best combination of comfort and cold with the newest ColdSPOT featuring a beautiful, feminine floral pattern design. Only Beekley offers the longest lasting cold therapy product on the market with attached adhesive so it stays in place without shifting. Advanced moldable gel allows for comfortable wear of the product throughout the course of treatment.

New ColdSPOT is not an ice-pack, but a moldable, comfortable gel pack that stays cold longer than current products, reducing bleeding, bruising, pain and inflammation.

Only ColdSPOT has attached adhesive tabs for exact positioning which eliminates shifting and the need for bulky bandages.

The New ColdSPOT also features beautiful, feminine floral designs.

New, Improved ColdSPOT - - perfect for post-biopsy cold therapy.

...and only available from Beekley

Perfect for stereotactic core or ultrasound breast biopsies  
Use before or after the procedure to help reduce bleeding, bruising, pain and inflammation.

#### Instructions for use:

1. Pre-freeze ColdSPOT for a minimum of 2 hours before use.
2. When needed, remove ColdSPOT and sterile gauze from outer packaging.
3. Apply sterile gauze dressing over the biopsy site.
4. Remove cover over the latex-free adhesive tabs on either side to keep it in place and apply ColdSPOT over dressing.
5. Use cold treatment as directed by physician.

ColdSPOT<sup>®</sup>  
Order Code: 502  
Contents: 68 ColdSPOTS



Patent 5,887,437

To place an order contact BEEKLEY today:  
1-800-BEEKLEY (233-5539) Fax: 1-800-735-1234 [www.beekley.com](http://www.beekley.com)